Augentropfen

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Similasan

TEARS AGAIN® Day

Similasan

TEARS AGAIN® Day

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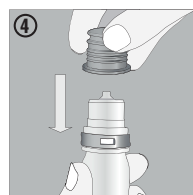
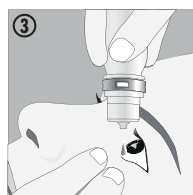
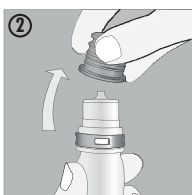
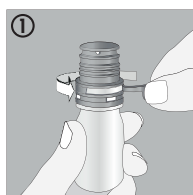
Ingredients

Sodium hyaluronate 1 mg/ml, sodium chloride, sodium hydrogen phosphate, chlorides (potassium, calcium, magnesium), sodium hydrogen carbonate, purified water.

Areas of application

For use by contact lens wearers to lubricate the eye surface in case of dry, irritated or burning eyes and for the relief and treatment of irritations due external factors such as a dry or air-conditioned environment, smoke, dust, pollen.

Similasan TEARS AGAIN® Day eye drops can also be used with hard or soft contact lenses.

Contraindications

Similasan TEARS AGAIN® Day eye drops should not be used in the presence of a hypersensitivity to any of the ingredients.

Notes

- Please note that your vision may be temporarily affected immediately following the use of the product.
- Do not use *Similasan TEARS AGAIN® Day* eye drops if the bottle has been damaged.
- Do not use *Similasan TEARS AGAIN® Day* eye drops simultaneously other eye medicines because the effect of the preparations may be altered. In principle, if other eye drops / ointments are used, an interval of approx. 30 minutes between using the different products should be adhered to.
- In order to avoid contamination of the external surface, please ensure that you avoid touching the surface of the eye or other items with the tip of the dropper.

Tips for use

Drop *Similasan TEARS AGAIN® Day* eye drops onto the surface of the eye or onto the conjunctival sac.

- Washing hands thoroughly before use is recommended.
- Remove the tamper-evident seal before first use (see Fig. 1).
- Remove the screw cap before each use (see Fig. 2).
- Tilt your head back a little (see Fig. 3).
- Hold the bottle vertically over each eye in turn, with the opening facing downwards (see Fig. 3).
- Carefully open your eye using the thumb and index finger of your free hand or gently pull the lower lid down (see Fig. 3).
- By gently squeezing on the bottle, 1 drop is instilled onto the surface of the eye or onto the conjunctival sac (see Fig. 3).
- Repeat the application for the other eye.
- Put the protective cap immediately after each use (see Fig. 4).

Directions for use

TEARS AGAIN® Day eye drops can be used as required. In general, application up to 4 times per day is recommended.

Side effects

Hypersensitivity reactions may occur in very rare cases. In this case, the product should not be used again.

Shelf-life and storage

Store in a cool and dry place. Keep out of reach of children. Can be used for up to 12 weeks after first use. Do not use after the expiry date.

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Can be used for up to 12 weeks after first use

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